

## Clinical Assistance Guide

### Aspire Vogue Carbon Fibre Seat Walker



# Aspire Vogue Carbon Fibre Seat Walker



The Aspire Vogue Carbon Fibre Seat Walker is a 4 wheel, lightweight walker with a cross folding frame, making it light and easy to use, store, and transport. Its European, ergonomic design offers personalised height adjustment that promotes correct posture, and the small turning circle offers excellent manoeuvrability and stability in tight spaces.



Now available in 3 sizes – Short (Sapphire blue frame), Medium (black frame), and Tall (Emerald green frame). It has a mesh seat and supportive backrest, allowing the user to rest when out and about. A stylish zip-top utility bag secures up to 5kg and easily unclips for shopping.

One of the lightest seat walkers on the market, the Aspire Vogue Carbon Fibre Seat Walker, with a super lightweight carbon fibre frame, provides premium function and elegant styling, for a superior walking experience.

## FEATURES

Super lightweight	<ul style="list-style-type: none"><li>• Carbon fibre frame to maximise portability</li></ul>
Luxury look and feel	<ul style="list-style-type: none"><li>• Danish design, profiled tubing, moulded fittings, ergonomic handles and refined colours</li></ul>
Ergonomic design	<ul style="list-style-type: none"><li>• Designed with a small turning circle for excellent manoeuvrability in tight spaces</li></ul>
AutoLocate	<ul style="list-style-type: none"><li>• Smooth height adjustment that automatically locates to pre-set heights, making adjustment simple and accurate</li></ul>
Simple folding mechanism locks	<ul style="list-style-type: none"><li>• For easy transportation and storage</li></ul>
Advanced locking handbrakes	<ul style="list-style-type: none"><li>• For increased user safety when walking and sitting</li></ul>
Unbreakable TPE super-soft wheels	<ul style="list-style-type: none"><li>• Offers high durability and a smooth ride for increased comfort</li></ul>
Removable utility bag	<ul style="list-style-type: none"><li>• Stylish and simple to remove for shopping. Zip-top bag secures up to 5kg of items</li></ul>
Safe Working Load	<ul style="list-style-type: none"><li>• 150kg</li><li>• Shopping bag – 5kg</li></ul>
Inclusions	<ul style="list-style-type: none"><li>• Supplied with bonus cane holder</li></ul>



## ACCESSORIES



Oxygen Bottle Holder Bag  
WAA695600



Night Light – LED Lamp  
WAA695630



IV Pole – Height Adjustable  
WAA695651



Single Hand Brake System  
WAA695640

## IDEAL USER CONDITIONS

The Aspire Carbon Fibre Seat Walker may assist those who require support with walking due to the following:

- Joint pain in the back, hips, knees, or feet due to arthritis, osteoporosis, or gout
- Back pain or spinal canal stenosis
- Reduced balance due to a neuromuscular condition such as:
  - Multiple Sclerosis
  - Parkinson's Disease
  - Motor Neurone Disease
  - Stroke
- Fatigue or need for frequent rest breaks
- Chronic Obstructive Pulmonary Disease and breathlessness

## CLIENT OUTCOMES

### Increased confidence and independence both inside and outside of the home:

- The lightweight, Aspire Vogue 4-wheel walker with its tight turning circle and good manoeuvrability, can support users to access areas they need to within their home
- The large wheels and lightweight frame make it easier to manage different floor surfaces and suitable outdoor terrain
- The enhanced stability and support provided to those with impaired mobility, can increase the user's confidence and feelings of safety to continue their daily activities

### Enhanced community access and social connection:

- The lightweight and easy folding frame makes the Aspire Vogue Carbon Fibre Seat Walker easy to lift into and out of a car, supporting access to the community and continued social engagement
- The seat and back support enable rest breaks when out and about, to assist pacing, energy conservation, and reduce pain and fatigue. This can increase confidence in going out as the user can be assured that they can rest when needed

## SPECIFICATIONS

Model	Overall Length	Overall Width	Overall Height	Seat Depth	Seat Width	Seat Height	Folded Width	Wheel Size	Minimum Functional Weight	Typical Functional Weight*	Bag SWL
	mm / in	mm / in	mm / in	mm / in	mm / in	mm / in	mm / in	mm / in	kg / lbs	kg / lbs	kg / lbs
Small	655 / 26	600 / 23.6	715 – 830 / 28 – 32	240 / 9.4	460 / 18	500 / 20	230 / 9	203 / 8	4.7 / 10.4	5.9 / 13	5 / 11
Medium	655 / 26	615 / 24	725 – 940 / 28 – 37	240 / 9.4	460 / 18	550 / 22	230 / 9	203 / 8	4.95 / 11	6 / 13.2	5 / 11
Tall	655 / 26	600 / 23.6	780 – 1030 / 30 – 40	240 / 9.4	460 / 18	620 / 25	230 / 9	203 / 8	5.2 / 11.5	6.35 / 14	5 / 11

\* Includes accessories, bag and backstrap.

**2 YEAR WARRANTY**  
Scooter Frame

**2 YEAR WARRANTY**  
Motors/Axles/Brakes

**6 MONTH WARRANTY**  
Battery

## COMPLIANCE

ISO 11199-2:2021

## MAINTENANCE

Weekly	<ul style="list-style-type: none"><li>• Check brake function</li><li>• Check all wheels are secure</li><li>• Check all fastenings such as nuts, bolts, and hinges are fastened correctly and functioning normally</li></ul>
Periodically	<ul style="list-style-type: none"><li>• Check hand grips for wear</li><li>• Check frame bolts are secure</li><li>• Check folding hinge bolts are secure</li><li>• Check front and rear axle bolts are secure</li><li>• Check for wear or cracks</li><li>• Check wheels for splitting or damage</li><li>• Check seat upholstery for wear and seat base for cracks and that it is secured to frame</li></ul>
Yearly	<ul style="list-style-type: none"><li>• Take your walker to an authorised repairer for a full check, clean, service, and lubrication of all moving parts.</li></ul>
Serviceable Components	<ul style="list-style-type: none"><li>• Wheels</li><li>• Brake Cables</li><li>• Bearings</li><li>• Bag</li><li>• Seat</li><li>• Nuts and Bolts</li><li>• Hand grips</li><li>• Brake Handles</li></ul>

## AIDACARE KNOWLEDGE HUB

### Aidacare Website Link:

- [Aspire Vogue Carbon Fibre Seat Walker | Aidacare](#)

### PDF Resources:

- [Aspire Vogue User Manual](#)
- [Aspire Vogue Walker Brochure](#)
- [Aidacare Choice Assistance Guide – Walkers](#)
- [Safe Use Guides - Walking Frames & Rollators](#)
- [Aspire Vogue Carbon Fibre Clinical Assistance Guide](#)

### Video Library:

- Aspire Vogue Walker Range: [VIMEO Vogue Carbon Fibre SeatWalker Video](#)