



# SWIVEL SHOWER CHAIR

## USER MANUAL



Thank you for choosing  
this Aspire Swivel Shower Chair.

This user manual contains a description of the product and important guidelines to ensure correct and safe use. It is important to read this manual carefully, and in its entirety prior to use.

**It is especially important to  
read the safety requirements and follow these.**

Aidacare continuously improves products and reserves the right to change the specifications and functions of products without notice.

If you have any queries, please contact your dealer or Aidacare directly. Contact information is located on the last page of this manual.

© Aidacare Pty Ltd 2026

No part of this user manual may be copied, distributed, reproduced, scanned, or stored on any electronic database, whether in whole or in part in any form or by any means.

Due care has been taken to ensure all the information contained in this user manual is correct at the time of printing. All measurements, pictures, colours, and weight capacities are to be used as a guide only. We reserve the right to modify the design or appearance of any product displayed in this user manual without prior notice.

# TABLE OF CONTENTS

---

1. Product Description	4
2. Symbols Used in this User Manual	4
3. Symbols Used on the Product	5
4. Warnings	6
5. List of Components	9
6. Assembly Instructions	10
7. Instructions for Use	12
8. Product Specifications	14
9. Cleaning and Maintenance	14
10. Compliance	14
11. Warranty	15
12. Recycling Information	15
13. Manufacturer	15

# 1. PRODUCT DESCRIPTION

---

The Aspire Swivel Shower Chair is designed to assist individuals with reduced mobility to sit safely while showering.

It features a lightweight frame, a swivel seat to assist with positioning, height-adjustable legs to accommodate different user heights, non-slip rubber feet for stability, and armrests and a backrest for additional support and comfort.

The chair is intended for indoor bathroom use only.

The Aspire Swivel Shower Chair is suitable for:

- Adults and older people
- Users with reduced balance, strength, or endurance

To support safe use, always ensure the chair is correctly assembled and stable before use, and do not exceed the specified maximum user weight.

# 2. SYMBOLS USED IN THIS USER MANUAL

---

The symbols below are used throughout this user manual and on the product to identify warnings and important information. It is very important for you to read them and understand them completely.



**WARNING!** Indicates a potentially hazardous condition/situation. Failure to follow designated procedures can cause either personal injury, component damage or malfunction. On the product, this icon is represented as a black symbol on a white triangle with a black border.



**ALWAYS!** These actions should be performed as specified. Failure to perform mandatory actions can cause personal injury and/or equipment damage. On the product, this icon is represented as a white infinity symbol on a black dot with a white border.



**DO NOT!** These actions are prohibited. These actions should not be performed at any time or in any circumstances. Performing a prohibited action can cause personal injury and/or equipment damage. On the product, this icon is represented as a white symbol with a black circle and black slash.


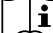

# 3. SYMBOLS USED ON THE PRODUCT

---

The manufacturer’s sticker shown below is a sample sticker only.



-  Product Code
-  Serial Number
-  Manufacturer
-  Medical Device

-  Safe Working Load
-  Read User Manual
-  Caution

## 4. WARNINGS

---

Safe Working Load (SWL)



DO NOT exceed the maximum user weight.



DO NOT stand on the shower chair.



DO NOT use the shower chair if it is unstable, damaged, or incorrectly assembled.



DO NOT use the shower chair for any purpose other than seated showering.




DO NOT modify this product in any way.

### Maximum User Weight 136KG/300LBS

(The weight needs to be evenly distributed on all four legs)

Store below 50°C

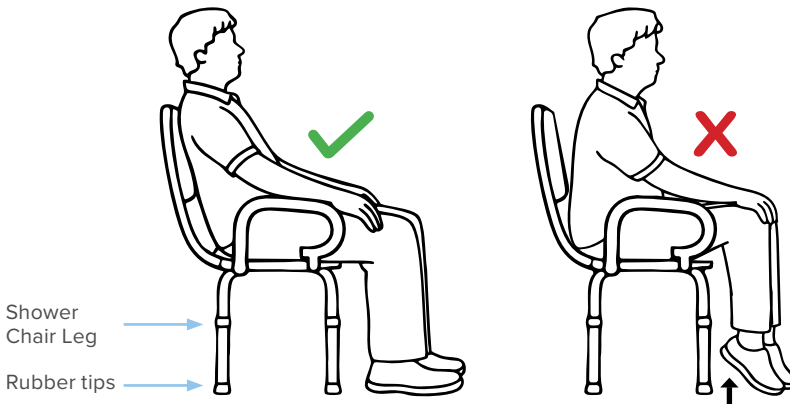

## Safe Setup and Stability



ALWAYS place the shower chair on a flat, stable, non-slip surface.



ALWAYS ensure all four legs are adjusted to the same height before use.



ALWAYS ensure all four legs are in full contact with the floor.



ALWAYS ensure all spring pins protrude fully through the adjustment holes.



ALWAYS test the stability of the shower chair before use.

## Safe Use



ALWAYS sit fully on the seat and distribute body weight evenly with feet flat on the ground.



ALWAYS ensure the seat is locked in position after swivelling and before sitting fully.



ALWAYS ensure armrests are securely locked in position before use.



DO NOT lean excessively to one side or sit on the edge of the seat.



DO NOT move or reposition the shower chair while it is occupied.

## User Supervision and Assistance



**ALWAYS** seek advice from a suitably qualified healthcare professional if unsure whether this product is appropriate for the user.



**ALWAYS** use supervision or assistance where the user has reduced balance, strength, or cognitive capacity.



**DO NOT** leave the user unattended if they require assistance.

## Environment and Accessories



**ALWAYS** ensure rubber feet are clean and in good condition before use.




**DO NOT** use the shower chair on uneven or sloped surfaces.



**ALWAYS** consider using a non-slip shower mat to reduce the risk of slipping.

## Safety Precautions

- Ensure all four legs of the shower chair remain in contact with the ground during use.
- Check the stability of the shower chair on uneven surfaces and adjust individual legs as needed for stability.
- Ensure that all four legs are adjusted to the same height.

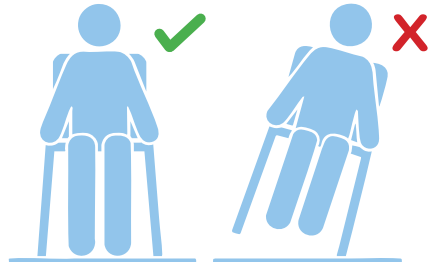

# 5. LIST OF COMPONENTS

---



- 1 Backrest x 1
- 2 Seat x 1

- 3 Foot Tube x 4
- 4 Swing-up Armrest x 2

# 6. ASSEMBLY INSTRUCTIONS

---

## 6.1 Initial Check

1. Remove the shower chair from the carton.
2. Check that all components listed in the list of components are present and undamaged.
3. If any parts are missing or damaged, do not assemble or use the product.

## 6.2 Assemble the Legs



1. Place the seat upside down on a stable surface.
2. Insert each assembled leg into the underside of the seat frame.
3. Ensure all spring pins are fully engaged and locked in place. Adjust all four legs to the same height.



### 6.3 Install the Backrest

1. Attach the backrest tubes to the backrest plate using the thumb screws.
2. Insert the backrest tubes into the seat frame until the spring pins pop out fully.
3. Select the preferred depth position.



### 6.4 Final Assembly Check

1. Turn the chair upright.
2. Check that all four legs are the same height. If not, adjust accordingly.
3. Ensure the chair is level and stable before use.



# 7. INSTRUCTIONS FOR USE

---

## 7.1 Initial Safety Check

Before each use:

- Ensure all spring pins are fully engaged and protruding out from the legs.
- Ensure all four legs are level and in contact with the floor
- Check the chair is stable and does not wobble
- Do not use the shower chair if it is unstable or damaged.

## 7.2 Adjust the Height

1. Adjust all four legs to the same height.
2. Ensure the user's feet rest flat on the floor when seated with their bottom fully supported.
3. Ensure all spring pins are fully engaged and protruding out from the legs after adjustment.


## 7.3 Using the Swivel Seat

- Lift and hold the bar located under the seat to enable rotation.
- Rotate the seat as required (up to 360°).
- Release the bar to lock the seat in the desired position.
- Ensure the seat is locked before sitting fully.



#### 7.4 Using the Armrests

- Press the red button to flip the armrest up for easier access.
- Lower the armrest until it locks securely before use.

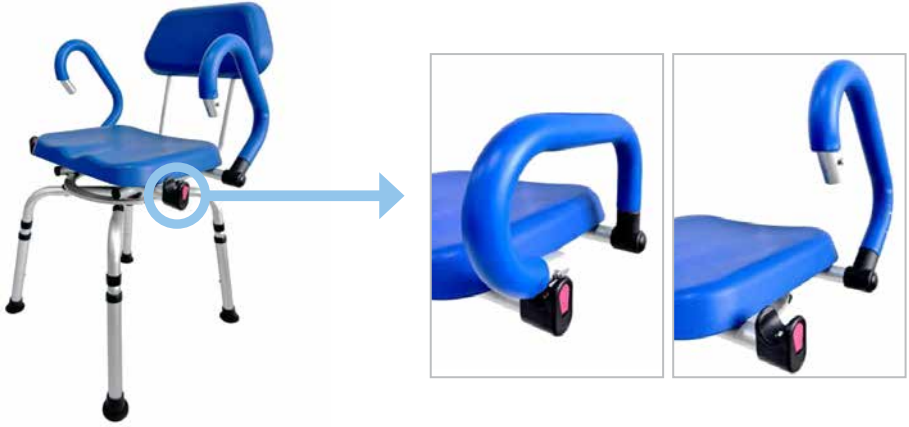

#### 7.5 Sitting and Standing

- Sit fully on the seat and distribute body weight evenly.
- Do not sit or perch on the edge of the seat.
- Do not stand on the shower chair.
- Assistance should be provided if required by the user's functional ability.
- Do not lean or unevenly distribute weight to one side of the shower chair.

## 8. PRODUCT SPECIFICATIONS

---

Overall Height	850 – 950mm
Seat Height (floor to seat)	510 – 610mm
Armrest Height (floor to armrest)	630 – 730mm
Seat Width (between armrests)	465mm
Seat Depth	420mm
Product Weight	5.85kg
SWL	136kg

All measurements are in millimeters unless otherwise specified. The safe working load capacity is tested with a dummy or personnel of the same weight.

## 9. CLEANING AND MAINTENANCE

---

- Regularly clean and sanitise the commode to maintain hygiene.
- Inspect the commode for any signs of wear or damage and replace any worn-out parts promptly.

Should you have any doubts about the correct functioning of the wheelchair or any of its components, please contact Aidacare.

## 10. COMPLIANCE

---

This shower chair is manufactured in ISO compliant production facilities in P.R China to strict quality control standards.

Aspire shower chairs are manufactured and independently tested to the market relevant compliance standards.

Labels on the shower chair address compliance requirements and users should familiarise themselves with these prior to use.

## 11. WARRANTY

---

**1** YEAR  
WARRANTY

This product comes with a warranty of 12 months. If you have any questions, you can contact Aidacare.

## 12. RECYCLING INFORMATION

---

The Aspire Swivel Shower Chair is made from metal and plastic components. The chair can be recycled via standard processes in your city of use.

## 13. MANUFACTURER

---

If you have any questions, you can contact:



Manufacturer:

Aidacare Pty Ltd ABN 40 134 398 833

Building 3A, 1 Moorebank Avenue,  
Moorebank NSW 2170 Australia

1300 133 120 | [product@aidacare.com.au](mailto:product@aidacare.com.au)

AIDACARE PTY LTD

Building 3A, 1 Moorebank Avenue,  
Moorebank NSW 2170  
Australia

1300 133 120 | [aidacare.com.au](http://aidacare.com.au)

MADE IN PRC

040526

